

TeleDart Pole Syringe

Instruction/Safety Manual



Thank you for purchasing the TeleDart Pole Syringe, designed for use by Animal Care Professionals.

Made from high quality unplasticized polyvinyl chloride (UNPVC) with a mechanical stability between 32—140°F. The pole syringe comes in two pieces, which can be joined together when a longer pole is needed (one piece is 3' long and the other is 1' long), you can also purchase additional 1' extensions to make the pole syringe even longer.


Included in the Pole Syringe Kit is one 3' and 1' Poles, Metal Plunger Rod, Luer Lock Syringe, Syringe Guard and a 19g x 1 1/2" needle. Pole Syringes are available in 3cc, 6cc and 12cc. All heads are interchangeable with the poles and can be purchased separately.

How to use:-

1. Connect the poles together if needed.
2. Pull the plastic plunger rod out of the syringe and remove the black rubber part from the end of the plastic plunger and place over the metal plunger of the pole.
3. Insert metal plunger into the syringe..
4. Push the red needle guard over the end of the syringe and then screw needle in.
5. Syringe can be loaded with medication prior to screwing onto the pole or when the syringe is connected to the pole.
6. Gently push the needle into the animal and slowly push on the pole to inject.
7. We recommend practicing using this pole syringe prior to use in the field.

Care & Safety:-

- The pole is for veterinary use only.
- Dispose of used needles in an approved "sharps" container.
- Do not try to bend the pole as this could result in the pole breaking.
- We recommend a damp cloth to clean the pole syringe —DO NOT USE BLEACH.
- Handle with care when using this piece of equipment.

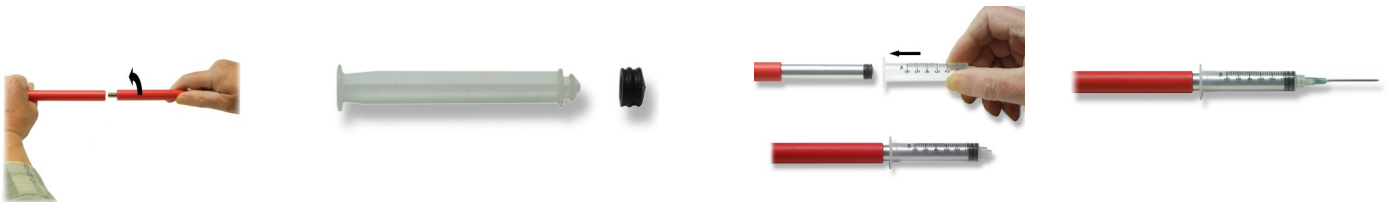

Animal Care Equipment & Services LLC

555 Alter St, 19A Broomfield CO 80020

T: 303 296 9287 | E: aces@animal-care.com | www.animal-care.com